HOFFMANN'S ABDRUCKMASSE grün

in Stangen

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HOFFMANN'S IMPRESSION COMPOUND green

sticks

Instruction for the dentist

Composition: Impression compound contains:

stearin, copal, talcum, wax, colour

Area of Application: copper-ring impressions, correction of models, raising of impression edges

Contra-indications: No known contra-indications

Side effects: No known side effects

Interactions with other substances: No known interactions

Processing Recommendation: Soften the quantity required by placing it in a water bath at 58 - 62°C for about 5 minutes. Remove the softened material from the water bath and immediately place it in the warmed copper ring. Before placing it into the mouth, briefly draw the side facing the cavity through a flame to smooth the surface. A film of glycerine or saliva on the surface of the tooth prevents the adhesion of the impression compound. The hardening may be accelerated by syringing with cold water or a cold air jet. After hardening is complete, remove the impression with a firm pull in the direction of the tooth's axis. The cold impression is then removed from the ring.

Setting Time: 1 - 1.5 minutes (at 37°C).

Package Size: Package with 12 sticks, 108 g

HOFFMANN'S Impression compound green

- For dental use only
- Do not use after the expiration date

HOFFMANN DENTAL MANUFAKTUR GmbH, Wangenheimstraße 37/39,
14193 Berlin, Germany, www.hoffmann-dental.com